



**SECURE
SHIELD**

Face Shield Disinfection and Cleaning Instructions



1 Wash hands with soap OR sanitize hands thoroughly.



2 Using gloves, detach visor piece from head frame. *(See product instructions)*



3 Apply hydrogen peroxide or 70% isopropyl alcohol solution to a soft cleaning cloth.



4 Gently, but thoroughly wipe both sides and edges of visor. *(Apply more solution to cloth as needed)*



5 Thoroughly wipe down frame piece. Be sure to go over all gaps and crevices.



6 Reattach visor piece to frame piece. *(See product instructions)*



7 Let air dry on a clean surface.



8 Properly dispose of gloves and cleaning cloth.



9 Wash hands with soap OR sanitize hands thoroughly.

- ▶ We recommend the above method of disinfection before each shift as part of routine maintenance.
- ▶ If face shield has any dust, debris or spatter, we recommend a deep clean to remove contaminants: Use warm water and a mild dish soap, OR a solution of 1 part white vinegar to 16 parts water. Gently wash all surfaces of visor with soap or vinegar solution. Allow to air dry.
- ▶ Additional Notes:
 - To remove streaks or spots, use standard household window/glass cleaner and a soft cloth.
 - Anti-Fog Wipes can be use on both sides of a clean visor if fogging is an issue.
 - Use of hydrogen peroxide may cause visor to become hazy over time.
 - Replace visor if damaged and/or if scratched or hazy and visibility is compromised.



Gerry Health Labs 
by Toronto MicroElectronics Inc.

www.HiGerry.com | 1.888.811.0132

6185 Danville Road, Mississauga, ON, L5T 2H7, Canada
1455 Alderman Drive, Alpharetta, GA 30005, USA

*We recommend disinfection before each shift as part of routine maintenance.
If face shield has noticeable dust, debris or, we recommend a deep clean first to remove solid contaminants.*

Use warm water and dishwashing detergent. Allow to air dry. To remove streaks or spots, use standard glass cleaner (e.g., Windex) and a soft cloth or paper towel.

You can use Anti-Fog Wipes to clean the shield both side when the shield become too fogging to use.

You can wipe the face shield with a solution comprised of 1 oz. white vinegar to 16 oz. warm water, or (2) wash the face shield with liquid dishwashing detergent. A thin film of soap left behind helps prevent fogging.

To disinfect your face shield, use 70% isopropyl alcohol solution or Hydro peroxide and a soft cloth, isopropyl alcohol wipes, to wipe both sides of the shield and head frame.

If you chose to wash the shield with warm water and soap, keep the shield in a bag for a minimum of 5 days.

Note: Hydrogen peroxide may cause visor to become hazy over time.